

FREQUENTLY ASKED QUESTIONS ABOUT USE AND MAINTENANCE OF SWIMMING POOL LIFT LEAQUATIC

1) How long do batteries work without being recharged ?

Batteries can grant up to 150 lifting movements , on the other hand the descending of the arm into water is granted by the opening of the valve which reduces the pressure on the lifting piston, so the arm goes down because of gravity.

2) Which is the max depth reached by the chair ?

The chair can go down up to 90 cm from the edge of the pool . If the level of water reaches the edge of the pool , the 90 cm can be too much, so it is better to stop before. But where water starts 30 cm down from the edge of the pool it means that the chair can go down for 60 cm into water and this is enough .

3) Which is the max capacity of the swimming pool lift ?

The swimming pool lift can reach the 140 kg capacity but because of homologation the max capacity has been reduced to 110 kg (load limit is 120 – 130 kg) .It is enough to put one foot on the ballast to avoid the possibility of overturning.

4) If the battery is out of charge and the movement of lifting up has not been completed, how can I manage to bring the person out of water ?

The lift is equipped with a manual emergency pump , you only have to take the crank which is inside the steering column, on the bottom part of it. Then you have to connect it with the electro hydraulic pump and then moving the crank the lifting arm will continue to work.

5) If there is an emergency while the person is going into – or out of water, how can I stop the lifting arm ?

There are 5 different safety systems, among them there is the red emergency button, you just have to press it and the lifting arm stops immediately .

6) How can I stop the swimming pool lift at the edge of the pool ?

You just have to use the brake lever , that is also a safety system. If the brake lever has not been activated the lifting arm does not work because of the blocking system to which the brake lever is connected.

7) If the front wheels of the swimming pool lift are beyond the edge of the pool how can I be sure that the lift will not fall into water ?

The swimming pool lift is equipped with two rubber feet that avoid the overturning of the machine.

8) How much time do batteries need for recharging ?

About 5 hours if they are completely out of charge , please take into consideration

that you can check the battery level using the battery level indicator on the dashboard. To assure a long life of batteries it is better to have them always charged even when you do not use the lift for a long time.

9) Is there a warranty for the swimming pool lift ?

The swimming pool lift has a 2 years- warranty for defects in materials and workmanship , apart from damages due to modifications or alterations caused by unauthorized persons. Batteries have a 6 months- warranty.

10) What about maintenance ?

The swimming pool lift needs no maintenance, it is enough to check the battery level , if the lift has been used a lot, a yearly check up made by authorized technicians can be useful.

11) If one wheel is punctured, how can I manage to replace it ?

The swimming pool lift is equipped with no puncture wheels so this is not going to happen. In any case our company grants the delivery of any spare part in the max time of 72 hours all over Europe , if you need the help of a technician we are going to send one as soon as possible.

12) The swimming pool lift has been damaged by a collision against a wall corner, and the paint has been scratched, how can avoid rust ?

The swimming pool lift has not only been painted but also galvanized, this means that it is really difficult to find rust, in any case the swimming pool lift is equipped with a little can with some paint for painting up. You just have to use sand paper on the damaged zone and then to re-paint it .

The swimming pool lift is also equipped with another safety system, two feet on the front side automatically lean upon the edge of the pool while the lifting arm goes down, avoiding any possibility of overturning.